

# VERRUCA-FREEZE®

Safe, Easy & Effective Cryosurgery  
www.verrucafreeze.com




## Did you know Verruca-Freeze treats 21 different types of skin lesions?

### COMMONLY TREATED LESIONS

#### Skin Tags (Acrochordons)

**Tips:** Often irritating but easy to treat, skin tags form primarily in areas where the skin forms creases, such as the neck, armpit and groin.

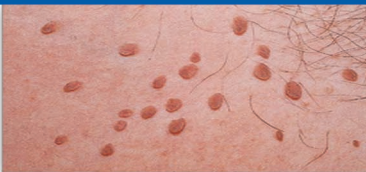

**Freeze Time:**  
35 - 40 Seconds

**CPT Code**  
11200 (1-15)

**Reimbursement**  
50<sup>th</sup> Percentile: \$139.91  
75<sup>th</sup> Percentile: \$174.27  
90<sup>th</sup> Percentile: \$220.58

#### Molluscum Contagiosum

**Tips:** This is a common pediatric condition. One saturated CryoBud can treat multiple lesions. These lesions are often 1-5 millimeters in diameter, with a dimpled center.



**Freeze Time:**  
35 - 40 Seconds

**CPT Code**  
17110 (1-14)

**Reimbursement**  
50<sup>th</sup> Percentile: \$127.55  
75<sup>th</sup> Percentile: \$172.22  
90<sup>th</sup> Percentile: \$229.77

#### Seborrheic Keratoses

**Tips:** Seborrheic keratosis is also known as Senile warts. It is seen more often in middle age patients, found in groups on the patients back.



**Freeze Time:**  
35 - 40 Seconds

**CPT Code**  
17000 (1)

**Reimbursement**  
50<sup>th</sup> Percentile: \$113.47  
75<sup>th</sup> Percentile: \$153.08  
90<sup>th</sup> Percentile: \$204.24

#### Actinic Keratoses (Solar Keratoses)

**Tips:** Actinic keratosis is very common, affecting half of the global population. It is skin damage caused by years of exposure to ultraviolet rays.



**Freeze Time:**  
40 Seconds

**CPT Code**  
17000 (1)

**Reimbursement**  
50<sup>th</sup> Percentile: \$113.47  
75<sup>th</sup> Percentile: \$153.08  
90<sup>th</sup> Percentile: \$204.24

#### Age Spots (Lentigo)

**Tips:** Age Spots are blemishes on the skin associated with aging and exposure to ultraviolet radiation from the sun often found on the hands and is easily treated.



**Freeze Time:**  
20 - 40 Seconds

**CPT Code**  
17000 (1)

**Reimbursement**  
50<sup>th</sup> Percentile: \$113.47  
75<sup>th</sup> Percentile: \$153.08  
90<sup>th</sup> Percentile: \$204.24

#### Genital Warts (Condylomata acuminata)

**Tips:** Highly contagious sexually transmitted disease caused by some sub-types of human papillomavirus.



**Freeze Time:**  
35 - 40 Seconds

**CPT Code**  
54056

**Reimbursement**  
50<sup>th</sup> Percentile: \$225.50  
75<sup>th</sup> Percentile: \$345.00  
90<sup>th</sup> Percentile: \$450.61

Reduce freezing time by 10-15 seconds for facial and thin skin lesions.

 **CRYOSURGERY, INC.**

5829 Old Harding Pike, Nashville, TN 37205 • 615.354.0414 • www.cryosurgeryinc.com



# VERRUCA-FREEZE<sup>®</sup>

Safe, Easy & Effective Cryosurgery  
www.verrucafreeze.com



**Verruca-Freeze treats all types of warts including the following:**

## COMMONLY TREATED Warts

### Common Warts (Verruca vulgaris)

**Tips:** Easy to treat, one saturated CryoBud can treat multiple lesions.



**Freeze Time:  
30 - 40 Seconds**

### Flat Warts

**Tips:** Most often, these lesions affect the hands or face, and a linear arrangement is not uncommon.



**Freeze Time:  
20 - 30 Seconds**

### Filiform Warts (Digitate wart)


**Tips:** Use tweezers to pull the lesions away from the skin and freeze the base of the lesions.



**Freeze Time:  
30 - 40 Seconds**

### Plantar Warts

**Tips:** Using the freeze thaw freeze procedure can help achieve maximum resolution.

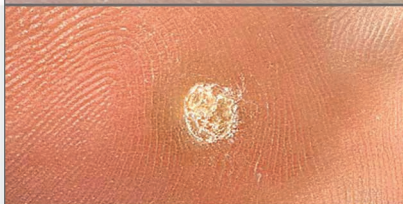

**Freeze Time:  
40 Seconds**

Allow a thaw time of approximately 40 seconds. We recommend freeze-thaw-freeze on lesions greater than 5mm and plantar warts.

	CPT: 17110
50 <sup>th</sup> Percentile	\$127.55
75 <sup>th</sup> Percentile	\$172.22
90 <sup>th</sup> Percentile	\$229.77

