

VERRUCA-FREEZE®

Safe, Easy & Effective Cryosurgery
www.verrucafreeze.com



Treat animals with warts, skin tags & other benign skin lesions!

COMMONLY TREATED LESIONS

Papilloma

Canine Papilloma Virus is a common virus in dogs as well as other mammals. Papillomas appear in a variety of shapes and sizes.



Freeze Time:
20 - 40 Seconds

Sebaceous Adenoma

Sebaceous Adenomas are a cutaneous condition characterized by a slow growing tumor. They are usually presented as a pink, flesh-coloured, or yellow papule or nodule.



Freeze Time:
20 - 40 Seconds

Skin Tag

Skin tags are small, benign tumors that form primarily in areas where the skin creases, such as the neck, armpit, and groin.



Freeze Time:
35 - 40 Seconds

Distichia

Distichia is an eyelash that arises from an abnormal spot on the eyelid of a dog and exits from the duct of the meibomian gland at the eyelid margin. They usually form in multiples and sometimes more than one arises from a single duct.



Freeze Time:
20 - 25 Seconds

Histiocytoma

Histiocytomas are an abnormal growth in the skin of histiocytes, a cell that is part of the immune system. They usually occur singly on the head, ears and legs.



Freeze Time:
20 - 30 Seconds

Sarcoid

Sarcoids are common, usually benign, skin tumors on horses and other equidae. They have several different forms.



Freeze Time:
30 - 40 Seconds

Other treatable lesions include, but are not limited to:
Follicular Cysts, Granulomas, Fibroids & More!

VERRUCA-FREEZE®

Safe, Easy & Effective Cryosurgery
www.verrucafreeze.com



Verruca-Freeze® is a convenient and cost effective alternative to traditional liquid nitrogen systems as well as scalpel removal because it is completely bloodless and requires no anesthetic, making it great for older pets.

Application

With two forms of application, treating benign skin lesions is now easier than ever! For delicate areas, we recommend using the CryoBuds® — simply spray onto a CryoBud® until it drips once, then place the bud onto the lesion for 25 seconds. For fleshy areas or large lesions, we suggest the CryoCones® — choose the cone that is 1 millimeter greater than the size of the lesion you are treating, place the cone over the lesion (rubber tip down) and spray into the inside wall of the cone for 3-6 seconds, wait for the cryogen to boil off completely, then remove the cone. After the procedure, allow the lesion to thaw for 45 seconds, and the process is complete. The lesion will blister slightly, scab, then fall off within 7-10 days.


BENEFITS

Portable
Self-Anesthetizing
Minimal Discomfort
No Suturing
2-Minute Procedure
No Evaporation
No Expiration
Low Cost Treatment

Tips For Treating Animals

Animals generally have thick skin, so debride thick, calloused warts before treatment. Use one freeze-thaw cycle for lesions on thin skin or lesions smaller than 5mm. Use two freeze-thaw cycles during one office visit for lesions 5mm or larger. The 12mm cone can be flipped to treat lesions that are 20-25mm.

Annual Revenue

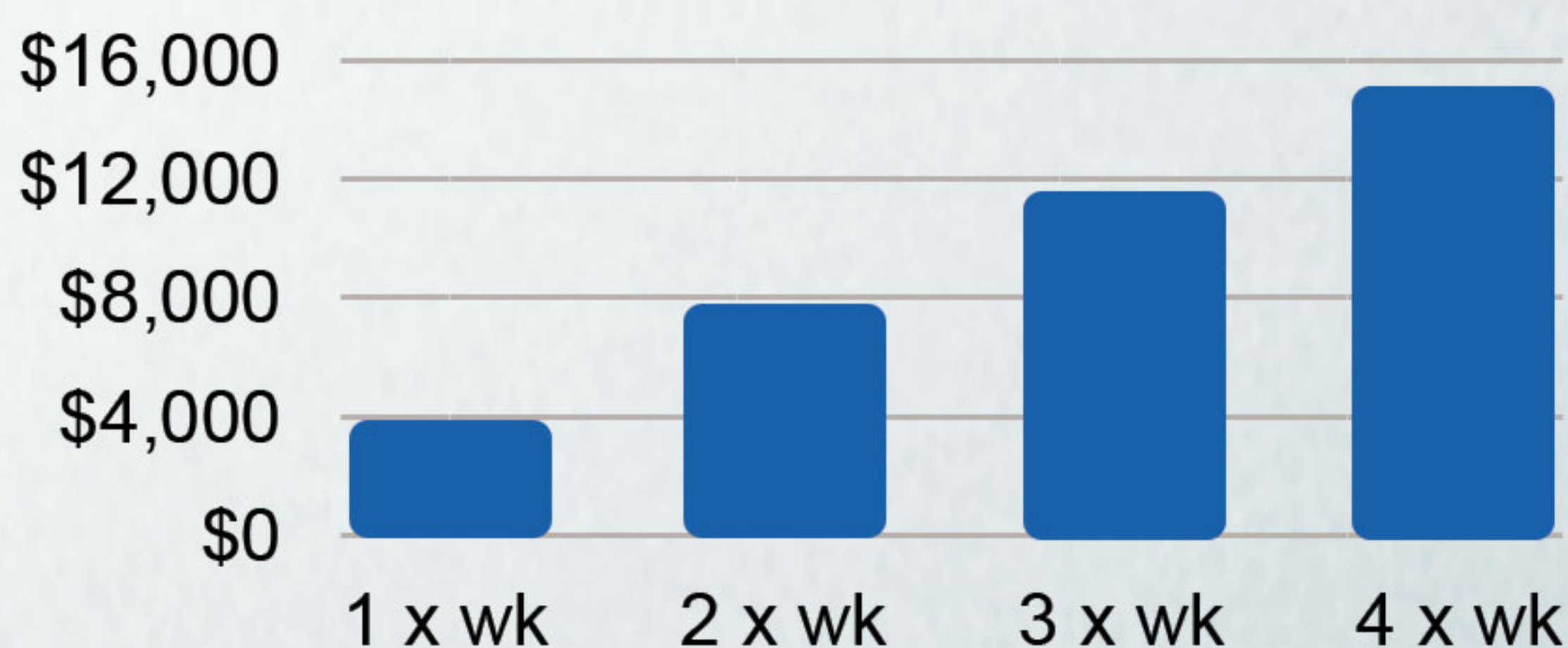

*Based on \$75 charge per treatment