

# VERRUCA-FREEZE

## CryoCone<sup>®</sup> Cleaning Instructions



### CDC Guidelines

According to CDC Guidelines for Disinfection and Sterilization, Verruca-Freeze<sup>®</sup> cones are classified as noncritical medical devices and require low-level disinfectant.

### Cleaning Options



#### EPA-Registered Disinfectant

Use a **low-level** or **intermediate-level** EPA-registered disinfectant.

Contact time must be at least **1 minute** to ensure disinfection.

Follow all applicable instructions on the disinfectant product label.

CaviCide<sup>™</sup> or CaviWipes<sup>™</sup> by Metrex are highly recommended.



#### Autoclave Machine

Use autoclave machine's default plastic setting (**121°C, 15 psig for 20 minutes**).

Cycle must be at least 15 minutes to ensure disinfection.

Refer to autoclave instructions for more information.

**Caution:** Autoclaving cones may reduce their shelf life.

**CryoCones<sup>®</sup> must be disinfected prior to initial use and after each use on a patient.**