

VERRUCA-FREEZE®

CryoCone® Cleaning Instructions



CDC Guidelines

According to CDC Guidelines for Disinfection and Sterilization, Verruca-Freeze® CryoCone® applicators are classified as noncritical medical devices and require low-level disinfectant.

Cleaning Methods



EPA-Registered Disinfectant

Use a **low-level** or **intermediate-level** EPA-registered disinfectant.

Contact time must be at least **1 minute** to ensure disinfection.

Follow all applicable instructions on the disinfectant product label.

CaviCide™ or CaviWipes™ by Metrex are highly recommended.



Autoclave Machine

Use autoclave machine's default plastic setting (**121°C, 15 psig for 20 minutes**).

Cycle must be at least 15 minutes to ensure disinfection.

Refer to autoclave instructions for more information.

Caution: Autoclaving cones may reduce their shelf life.

CryoCone® applicators must be disinfected after each use to prevent cross-contamination between patients.

